

ANGIODYNAMICS®

INCORPORATED

Perchik Button™

ANGIODYNAMICS, INC.
603 QUEENSBURY AVE.
QUEENSBURY, NY 12804 U.S.A.
TOLL FREE: 800-772-6446
PHONE: 518-798-1215
FAX: 518-798-1360

INDICATION FOR USE:

The Perchik Button™ is intended to aid wound healing by distributing suture tension over a larger area on the patient.

CAUTION:

Federal (USA) law restricts this device to sale by or on the order of a physician. The Perchik Button™ is supplied sterile and is intended for single use only. Do NOT re-use or re-sterilize this device.

The Perchik Button™ should be used by physicians, who have a thorough understanding of percutaneous interventional procedures.

Do not use if the package is damaged, opened or the expiration date has passed.

PRECAUTIONS:

During placement and removal of the Perchik Button™, follow universal precautions for exposure to blood and bodily fluid.

POTENTIAL COMPLICATIONS:

Potential procedural complications include, but are not limited to:

| | |
|-----------------------|---------------------|
| Allergic Reaction | Bruising |
| Cell Damage | Death |
| Drug Therapy | Drug Reaction |
| Edema | Embolism |
| Fever | Hematoma |
| Infection | Pain and Tenderness |
| Patient Discomfort | Septicemia |
| Surgical Intervention | |


PROCEDURE:

Placement

1. Use sterile technique.
2. Place Perchik Button™ over entry site with the flat side contacting the skin.
3. Draw suture tails through the holes on the button.
4. Tie tails with the knot located over the crevice on the top of the Perchik Button™.

Removal

1. Place scissors or scalpel with the sharp edge of the blade pointing upwards beneath the suture knot.
2. Cut the suture. If using a scalpel, cut the sutures by drawing the blade up through the knot in a motion moving away from the patient. Refer to the drawing below.

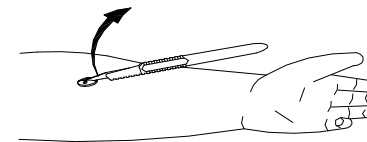

AngioDynamics® is a registered trademark of AngioDynamics, Inc.
Perchik Button™ is a trademark of AngioDynamics, Inc.
© AngioDynamics, Inc. 2004 all rights reserved

IC185 Rev. A