
## RITA® StarBurst™ Access System, Hard Tissue

**INDICATIONS:**  
To provide access for delivery of the RITA StarBurst devices, fluids, and material (such as saline) to interstitial tissue.

### INSTRUCTIONS FOR USE:



**Placement of the Introducer**

- When using the introducer be sure to place the stylet inside the targeted area with the stylet tip positioned where the tip of the StarBurst device will be.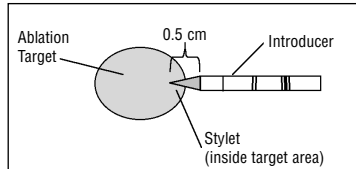

### Placement of the Device Inside the Introducer

- Place the StarBurst device **slowly** into the introducer.
- The device should go through the introducer **very** easily.
- The uninsulated portion of the trocar of the StarBurst device should be outside/distal to the introducer prior to deploying or starting an ablation.
- When using an introducer "To Hub" (see table), inset the StarBurst device into the introducer until the hub of the StarBurst device touches the proximal end of the introducer.
- If inserting a StarBurst device only partially into an introducer, insert the StarBurst device into the introducer until reaching the appropriate centimeter mark on the StarBurst trocar according to the table below

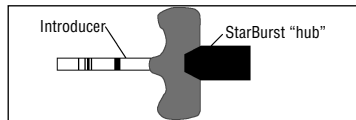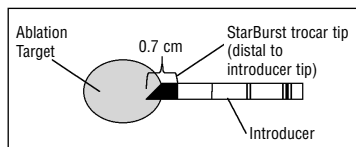Introducer/StarBurst Device Compatibility	Hard Introducer	
	6 cm (700-102330)	11 cm (700-102331)
<b>StarBurst Devices</b>	<b>To Hub</b>	<b>Not compatible</b>
<b>XL (10 cm)</b>	<b>To 10 cm mark</b>	<b>To 15 cm mark</b>
<b>XL Semi-Flex (25 cm)</b>	<b>To 10 cm mark</b>	<b>To Hub</b>
<b>XL (15 cm)</b>	<b>To 10 cm mark</b>	<b>To 15 cm mark</b>
<b>XL (25 cm)</b>	<b>To 10 cm mark</b>	<b>To 15 cm mark</b>
<b>XLI &amp; XLie (12 cm)</b>	<b>To 10 cm mark</b>	<b>To 15 cm mark</b>
<b>XLI &amp; XLie (25 cm)</b>	<b>To 10 cm mark</b>	<b>To 15 cm mark</b>
<b>Talon (15 cm)</b>	<b>To 12 cm mark</b>	<b>Not compatible</b>
<b>Talon (25 cm)</b>	<b>To 12 cm mark</b>	<b>To 17 cm mark</b>
<b>Talon Semi-Flex (25 cm)</b>	<b>To 12 cm mark</b>	<b>Not compatible</b>

Note: If there is any resistance on deployment, ensure that the StarBurst device is properly inserted into the point indicated above.

### Precautions

- CAUTION:** Do not attach anything abrasive (e.g., metal clamps, etc.) to the Introducer and/or Device. This may damage the insulation, which could contribute to patient injury.
- CAUTION:** Care must be taken when inserting the introducer to hard and/or abrasive tissue. This may damage the insulation, which could contribute to patient injury.

**P** This product is covered by U.S. and foreign patents and patents pending.

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