


# *LifePort Infusion Sets*

NON-CORING NEEDLES

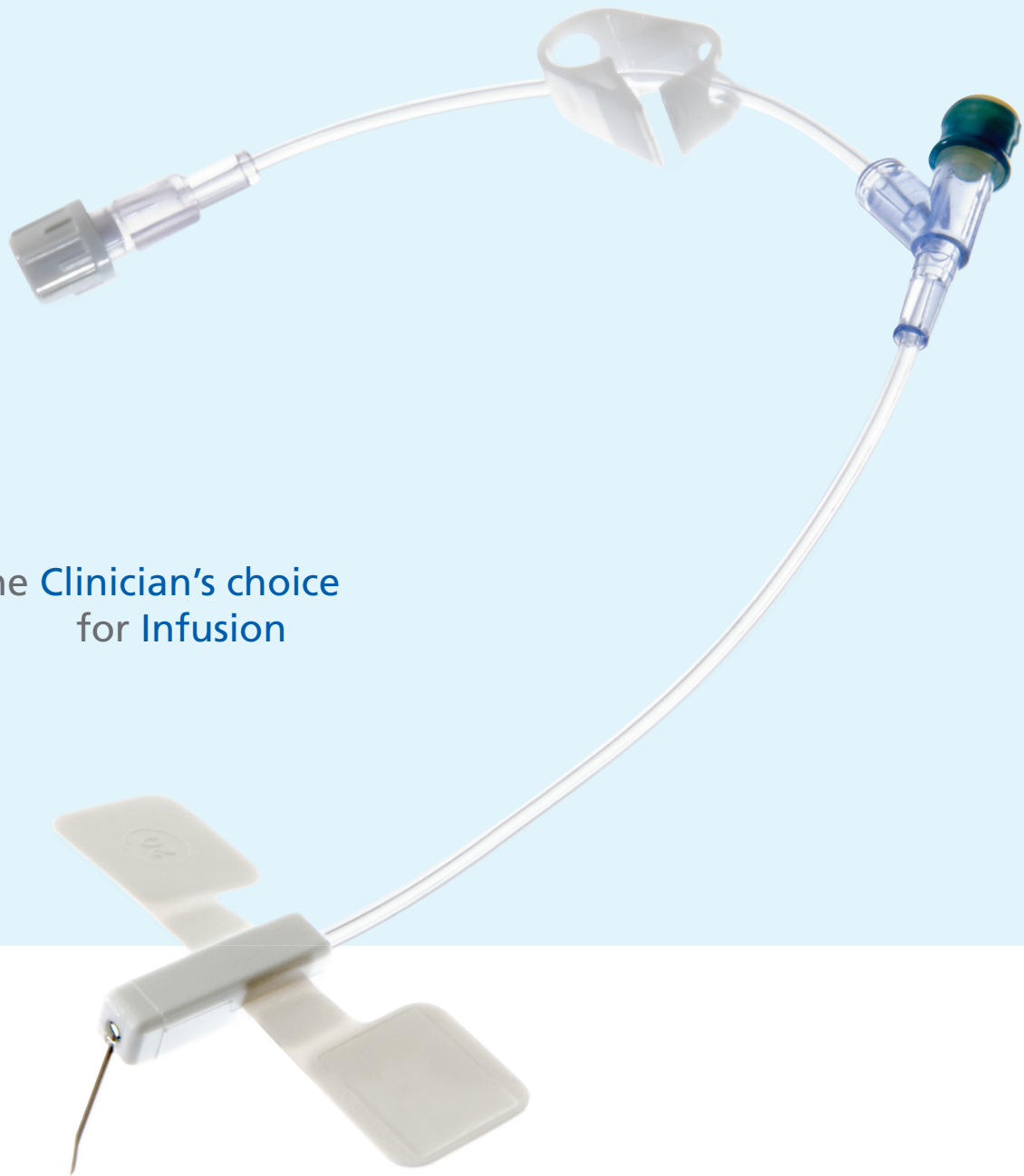

the Clinician's choice  
for Infusion

# LifePort Infusion Sets

## One Device for All Ports

AngioDynamics' LifePort® Infusion Sets are designed for use with all implantable ports. Designed for clinicians, LifePort Infusion Sets offer large grip wings for secure and maximum control.



## Benefits

- Easy to grip, place and control
- Easy to use with all implantable ports
- Larger grip handle for secure control
- Lipid-compatible
- Latex-free
- Height-adjustable wings improve needle support and facilitate securing the device to the patient regardless of the patient's size
- Easy to secure
- Anti-coring needle
- One-handed clamp

### LifePort Infusion Sets with Y-Site/Quantity (20 units/box)

Product Description	UPN
19 Ga x 1"	H787LPS30065
20 Ga x 1"	H787LPS30095
20 Ga x 1.5"	H787LPS30175
20 Ga x .75"	H787LPS30235
22 Ga x 1"	H787LPS30075
22 Ga x .75"	H787LPS30255

### LifePort Infusion Sets without Y-Site/Quantity (20 units/box)

Product Description	UPN
19 Ga x 1"	H787LPS30045
20 Ga x 1"	H787LPS30085
22 Ga x 1"	H787LPS30055

## IMPORTANT RISK INFORMATION

**INDICATIONS FOR USE:** The LifePort Infusion Set is used to access implanted vascular ports to administer fluids and/or to withdraw blood.

**CONTRAINDICATIONS:** Do not use with patients showing signs or symptoms of infection at the insertion site. Do not use with patients who have exhibited prior intolerance to the materials of construction.

**WARNINGS:** Maximum pressure recommended is 40 psi. A 10 ml syringe or larger is recommended. Do not use the

device if the package has been opened or damaged. Clamp infusion set when not in use. Store in a cool dry place.

**PRECAUTIONS:** Single Use Only: Non-Pyrogenic, Non-toxic. Contents sterile unless package is opened or damaged. Do not re-sterilize. Carefully read and follow all instructions prior to use of the device. Follow CDC guidelines, institutional guidelines or INS and ONS guidelines for infusion set use. Always maintain universal precautions and utilize aseptic technique throughout insertion care and

maintenance procedures. Application of excessive force to needle while accessing port may result in damage to needle and port. Keep hand /fingers protected from needle tip at all times by following the instructions for use.

Refer to product IFU for full warnings, precautions, possible adverse effects and contraindications.

**CAUTION:** Federal Law (USA) restricts this device to sale by or on the order of a physician.



USA > 14 Plaza Drive, Latham, NY 12110 > tel: 800-772-6446 or 518-798-1215 > fax: 518-798-1360  
International > Haaksbergweg 75 (Margrietoren), 1101 BR, Amsterdam Z-O > The Netherlands  
tel: +31 (0)20 753 2949 > fax: +31 (0)20 753 2939

[www.angiodynamics.com](http://www.angiodynamics.com)

\*AngioDynamics, the AngioDynamics logo and LifePort are trademarks and/or registered trademarks of AngioDynamics, Inc., an affiliate or a subsidiary.

© 2014 AngioDynamics, Inc. MLC 397 US Rev B