
## INSTRUCTIONS FOR USE

### Preparation:

- Remove the paper spacer from the trocar styllet (Fig. 1).

### Figure 1 – Drainage Catheter as Furnished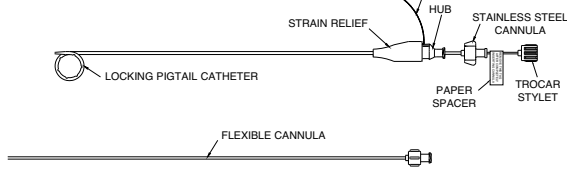


- Advance the stiffening cannula of choice (metal or plastic) into the drainage catheter and engage the luer fittings.

### Entry:

#### FOR TROCAR ENTRY TECHNIQUE:

- Advance the trocar inside the metal stiffening cannula and engage the luer lock fittings (Fig. 2).

### Figure 2 – Catheter Assembled for Insertion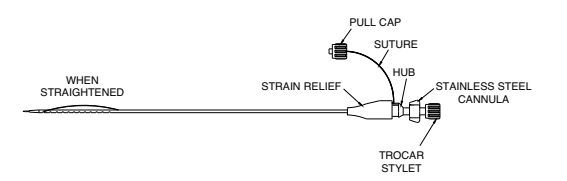

- Advance the assembly into the cavity.
- Remove the trocar.

#### FOR SELDINGER ENTRY TECHNIQUE:

- Optional: A guidewire may be placed through the stiffening cannula to aid in placement of the catheter in the cavity.
- Advance the catheter into the cavity by holding the stiffening cannula while advancing the catheter.


### Positioning:

- Under fluoroscopic guidance, form the locking pigtail by slowly removing the guidewire or stiffening cannula and rotating the catheter counterclockwise.
- To tighten the pigtail shape, gently pull the suture until resistance is felt.

### Engaging the Lock:

- Move the strain relief and slide to the locked position. (Fig. 3A to 3B) A ‘click’ will be heard when the lock is engaged.

### Figures 3A–3B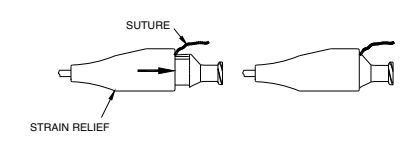


- Remove excess suture material and the pull cap by cutting the suture near the hub.

### Removal:

#### Option 1 – Unlocking the Hub

- Disconnect the drainage bag from the catheter.
- Move the strain relief distally, to expose the slide (Fig. 4A).

### Figures 4A–4C

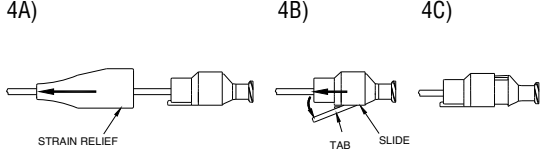- Lift the tab on the slide (Fig. 4B) and simultaneously move the slide to the unlocked position (Fig. 4B to 4C).
- If access is to be maintained, insert the recommended guidewire through the catheter. This may facilitate removal of the catheter while maintaining access.
- Gently withdraw the catheter.

#### Option 2 – Cutting the Catheter

- Disconnect the drainage bag from the catheter.
- Insert the recommended guidewire past the distal tip of the catheter.
- Carefully, cut the catheter in half just distal to the hub, ensuring that the suture is severed.
- Remove the hub from the guidewire.
- Gently remove the remaining distal portion of the catheter.

### PRODUCT DESCRIPTION

6.5F catheter containing multiple side holes, metal trocar, metal stiffening cannula, suture retention with pull cap and strain relief.

8F, 10F, 12F and 14F catheter containing multiple side holes, metal trocar, metal stiffening cannula, flexible plastic stiffening cannula, suture retention with pull cap and strain relief.

### INDICATIONS FOR USE

The catheters are designed for percutaneous drainage of fluids. This device is not indicated for direct contact with the heart, central circulatory system or the central nervous system.

### POTENTIAL ADVERSE EFFECTS

The following adverse reactions have been reported with the use of drainage catheters:

|                           |                        |
|---------------------------|------------------------|
| Biocompatibility Reaction | Bronchopleural fistula |
| Hemorrhage                | Loculation             |
| Sepsis                    | Skin Infection         |
| Pneumothorax              | Catheter Occlusion     |
| Pyopneumothorax           | Catheter Dislodgment   |
| Empyema                   |                        |

### WARNINGS

**Do NOT use this catheter with alcohol.**

**Do NOT use this catheter as a delivery system for nutritional supplements.**

Reuse of single-use devices creates a potential risk of patient or user infections. Contamination of the device may lead to injury, illness or death of the patient.

Reprocessing may compromise the integrity of the device and / or lead to device failure.

### CAUTION

Federal (USA) law restricts this device to sale by or use under the order of a physician.

This device is sterilized by ethylene oxide. The device is sterile and intended for single patient use. Sterile unless the package is opened or damaged. Do not re-sterilize.

Where long-term use is indicated, it is recommended that indwelling time not exceed 90 days. The physician should evaluate the catheter on or before 90 days post-placement.

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